

# XtremityTT™ FLEX Sizing Chart



Choosing the correct size XtremityTT FLEX Preform is critical for optimal fabrication and fit. The XtremityTT FLEX offers three different sizes (XtremityTT offers seven sizes) and all can be fabricated with the XtremityTT Socket System. Once the XtremityTT FLEX has been pulled over the socket, take measurements over the FLEX to determine sizing for the XtremityTT Socket.

**1**

## Distal Circumference

Circumferential measurement 4 cm from distal end of the limb model to determine the XtremityTT FLEX size.

**2**

## Length

Length measurement from MPT to distal end.

**3**

## Proximal Circumference

Circumferential measurement at MPT. Use the length and MPT circumference to ensure that the model fits within the XtremityTT FLEX sizing limits.

<b>1</b> → Size	SIZE 24	SIZE 28	SIZE 33
Distal Circumference:	Distal 24-27.9cm	Distal 28-32.9cm	Distal 33-38cm
<b>2</b> ↓ Limb Length: MPT to Distal End (cm)	<b>3</b> → MPT Circumference (cm)		
	Standard	Standard	Standard
7	26.9 – 34.9	28.5 – 37.0	32.5 – 42.3
8	27.2 – 35.4	28.7 – 37.3	32.7 – 42.6
9	27.5 – 35.8	29.0 – 37.6	32.9 – 42.8
10	27.9 – 36.2	29.2 – 38.0	33.1 – 43.1
11	28.2 – 36.7	29.5 – 38.3	33.3 – 43.3
12	27.6 – 30.5	29.7 – 38.6	33.5 – 43.6
13	28.9 – 37.5	29.9 – 38.9	33.7 – 43.9
14	29.2 – 38.0	30.2 – 39.3	33.9 – 44.1
15	29.6 – 38.4	30.4 – 39.6	34.2 – 44.4
16	29.9 – 38.9	30.7 – 39.9	34.4 – 44.7
17	30.2 – 39.3	30.9 – 40.2	34.6 – 44.9
18	30.6 – 39.7	31.2 – 40.5	34.8 – 45.2
19	30.9 – 40.2	31.4 – 40.9	35.0 – 45.5
20	31.2 – 40.6	31.7 – 41.2	35.2 – 45.7
21	31.6 – 41.1	31.9 – 41.5	35.4 – 46.0
22	31.9 – 41.5	32.2 – 41.8	35.6 – 46.2
23	32.3 – 41.9	32.4 – 42.1	35.8 – 46.5
24	32.6 – 42.4	32.7 – 42.5	36.0 – 46.8
25	31.9 – 42.8	32.9 – 42.8	36.2 – 47.0
26	33.3 – 43.3	33.3 – 43.3	36.2 – 47.3
27	33.6 – 43.7	33.6 – 43.7	36.6 – 47.8